

## FITTING GUIDE

### Vue Fitting Protocol

#### STEP 1.



Select the appropriate receiver cable length using the measurement tool in the fitting kit.



#### STEP 2.


Assemble the Vue device:

- Use the microphone cover removal tool to lift the microphone cover making sure that the microphone cover legs are completely raised out of the device.
- Plug the receiver cable into the device. It should sit flush with the case.
- Place the orange Real Ear microphone cover for the Real Ear measure or chrome microphone cover. Insert the narrow end first. Then snap the top portion down locking the receiver into the device. Following this measure, replace the orange microphone cover with the chrome microphone cover.

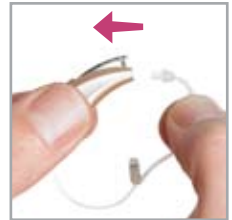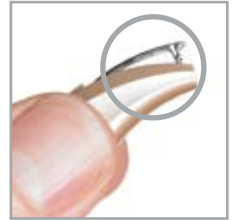

### STEP 3.

Plug the programming cable into the battery compartment of the device.

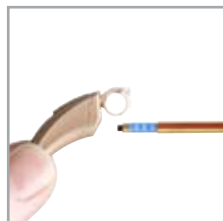

### STEP 4.

#### Receiver Matrix Selection

Inspire® OS prompts you to select the receiver gain level. The receiver recommendation is based on the audiometric configuration of the patient.



### STEP 5.

#### Vue Auto Path


Select the Acoustic Option when prompted.

This wizard provides an automatic, efficient method to complete an initial fitting. Auto Path steps vary based on device features.



You may choose to complete any or all of the Auto Path steps. Clicking on any orange icon removes that step from Auto Path and establishes a system preference. Select that icon again to enable the Auto Path step.



Auto Path provides step-by-step instructions. The current step is indicated by the Active icon.  Completed steps are indicated by both text and graphical display.

On devices with Real Ear, it is critical to remove all real ear accessories and place the chrome microphone cover prior to proceeding to the next step.

### STEP 6. Integrated Speech Mapping

Integrated Speech Mapping is an optional verification step for all Vue devices. It requires external speakers for self calibration.